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**IMPORTANT:** The Posture Correction T-shirt is a medical device. Please read these instructions for use carefully before the first use. Keep them for future reference.

## 1. Identification of the device and manufacturer

- Device name: Posture Correction T-shirt
- Manufacturer: eplecy sp. z o.o., Owocowa 3, 43-516 Zabrzeg, Poland, E-mail: [contact@eplecy.pl](mailto:contact@eplecy.pl), Website: [eplecy.pl](http://eplecy.pl)
- The device is non-sterile.
- Reusable device. Intended for a single patient.
- The Unique Device Identifier (UDI) code is located on the packaging.

## 2. Intended purpose and description of the device

The Posture Correction T-shirt is a posture corrector in the form of clothing that counteracts slouching. Supports an upright posture. Pulls shoulders back, opens the chest, and relieves the spine. Helps reduce back and neck pain caused by slouching. Unlike compression wear, it doesn't squeeze but supports the activation of your muscles.

### Intended purpose:

A medical device intended for the correction of body posture by supporting the restoration of the physiological alignment of the spine in the sagittal plane, and for the relief of back and neck pain associated with incorrect posture (e.g., during prolonged sitting).

### Method of operation:

The design of the device provides resistance when an incorrect posture (slouching) is adopted. This is intended to remind the user of the need to correct their posture by activating the appropriate muscle groups. When the posture is correct, the device's action is neutral.

### Indications:

The device is indicated in cases requiring posture correction, such as:

- Functional postural defects (e.g., round back, functional thoracic hyperkyphosis, shoulder protraction).
- Back pain (in the thoracic and/or cervical region) caused by incorrect posture.
- Weakness and/or muscle imbalance in the back and shoulder area associated with postural defects.

### Intended patient population and users:

The device is intended for adults, adolescents, and children over 7 years of age.

### Clinical benefits:

Using the device in accordance with its intended purpose and instructions contributes to increased awareness of correct posture (re-education) and to the reduction of discomfort and pain associated with maintaining an incorrect position for a long time.

The device is intended for self-use and does not require combination with other devices or accessories. The device is not intended for any purposes other than those indicated.

### 3. Contraindications

Do not use the device without consulting a doctor or physiotherapist in case of:

- Severe or fixed postural defects (such as advanced structural scoliosis) or spinal disorders.
- Recent injuries, inflammation, or postoperative conditions in the shoulder, neck, or trunk area.
- Skin diseases, open wounds, or epidermal damage at the point of contact between the device and the body.
- Circulatory, sensory, or respiratory disorders.
- Known allergy to any of the device's materials.
- Acute spinal pain or conditions of unknown etiology.

### 4. Safety of use

#### WARNING!

- The device is a supportive tool and does not replace physical activity or rehabilitation.
- Do not use the device while sleeping.
- Use in children only under adult supervision. It is not recommended for use in children under 7 years of age without consulting a doctor or physiotherapist.
- Use the device only for its intended purpose.
- Using the device may cause fatigue in postural muscles, which is a natural sign of their activation.

### PRECAUTIONS

- Choose the correct size. An incorrect size may cause discomfort, abrasions, or ineffective performance. Particularly in the women's model, a size that is too small may cause undesirable pressure on the bust.
- Consultation: If in doubt about the use of the device or your health condition, consult a doctor or physiotherapist.

Discontinue use and consult a specialist if:

- Pain, numbness, tingling, a rash, or other concerning symptoms occur while using the device.
- The ailments for which the device is used worsen or do not subside.

### Residual risk

The manufacturer has implemented a risk management system compliant with the requirements of Regulation (EU) 2017/745 (MDR) to identify and minimise all potential risks. When the device is used in accordance with its intended purpose and instructions, the residual risk is assessed as acceptable. It may include:

- Temporary skin discomfort, abrasions, or irritation if the size is improperly fitted.

## 5. Instructions for putting on and fitting

1. Choose the size according to the manufacturer's size chart. A correctly selected device should be fitted to the body but must not cause excessive pressure or restrict movement.
2. Put it on like a standard t-shirt. The easiest way is to start by putting your arms through the sleeves.
  - a. **Men's and children's model:** Ensure that the arms go through the inner, elastic straps in the sleeves.
  - b. **Women's model:** The inner, elastic band should be positioned under the bust. Excessive, bothersome pressure on the bust is a sign that the size is incorrectly chosen and a larger one should be selected.
3. The device has no adjustment elements. Its effectiveness depends on the correctly chosen size.

## 6. Recommended method of use

Gradually accustom your body to the device's action. Start by wearing it for 15-30 minutes, 2-3 times a day. This time can be gradually extended, observing the body's reaction. Remove the device and take a break when you feel muscle fatigue.

## 7. Cleaning and maintenance

To maintain the properties and durability of the device, follow the recommendations below:

**We recommend hand washing.** This is the best method to guarantee the device's effectiveness.

### Assessment of wear and tear:

Regularly inspect the condition of the device. Discontinue use and replace the device with a new one if there is visible mechanical damage or if it has visibly lost its original elasticity, which may affect its performance and is manifested by a significant reduction in resistance when slouching.

### Expected lifetime:

With proper care, the device should last for up to 12 months. This period depends on the intensity of use, individual user characteristics (e.g., perspiration), and adherence to the care instructions. In the case of very intensive use, this period may be significantly shorter. Regardless of the time elapsed, a regular functional assessment of the device, as described above, is key.

## 8. Storage conditions

Store in a dry and clean place, protect from sunlight and high temperatures.

## 9. Disposal

Dispose of the used device in accordance with locally applicable waste disposal regulations. The device should be treated as mixed waste, unless local regulations state otherwise.

## 10. Reporting of serious incidents

Any serious incident that has occurred in relation to the device should be reported to the manufacturer ([mdr@eplecy.pl](mailto:mdr@eplecy.pl)) and to the competent authority of the Member State in which the user and/or patient is established.

*(A serious incident is an incident that directly or indirectly led, might have led or might lead to any of the following: the death of a patient, user or other person, the temporary or permanent serious deterioration of a patient's, user's or other person's state of health, a serious public health threat).*

The manufacturer conducts continuous post-market surveillance to ensure the safety of the device.

## 11. Material composition

**Outer Layer:** 92% Cotton, 8% Elastane

**Inner Zone:** 85% Polyester, 15% Elastane

**Inner Mesh:** 92% Nylon, 8% Spandex

All materials used are certified with Oeko-Tex Standard 100 or have a safety attestation.

## 12. Explanation of symbols

LOT number

Date of manufacture

Manufacturer

Catalogue number

Consult instructions  
for use

Medical Device

Unique Device  
Identifier

CE marking indicating  
conformity with EU  
legislation

Packaging recycling  
symbol

Warnings /  
Contraindications

Triman symbol  
(required on the  
French market)

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